

RAYPEX® 6

NEW!

High-tech apex locator for precise length determination

Sets new standards with
respect to user-friendliness
and design


- Unique 3-D style colour touch screen
- Smart user interface
- “Plug-and-start” and “plug-and-check” features provide visual cues for enhanced ease of use
- Rechargeable battery
- Foldable, pocket-sized design





Precise length determination

- Latest multi-frequency apex locator technology
- Automatic calibration
- Separate RAYPEX® Apical Zoom shows the enlarged section between apical constriction and apical foramen


Coloured Apical Zoom

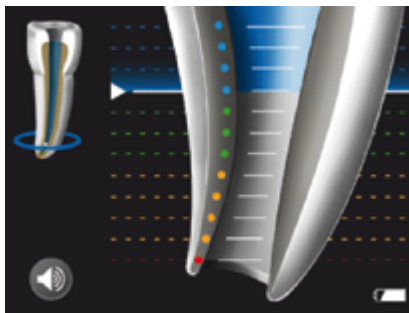

Blue section: approaching the apical region

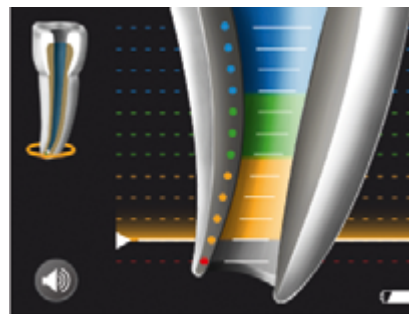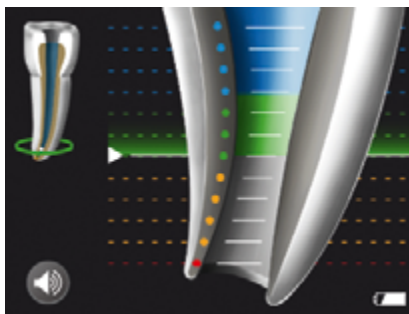

Green to yellow section:
projection of the region showing apical constriction to apical foramen - relevant section to determine the working length

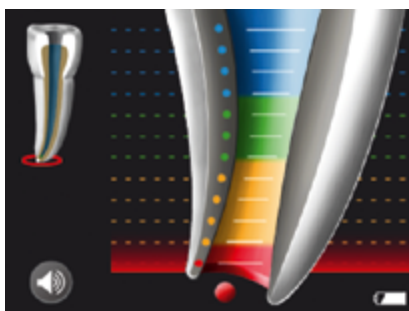

Red warning dot:
Case of over-instrumentation as measuring file has passed the apical foramen.

User-friendly interface

- Unique colour touch screen for smart operation
- Intuitive menu for setting options



- DEMO mode for clear information to patient
- Built-in CHECK mode for smooth functional test of device and cables, in case of questionable measuring result:
Automatic “plug-and-check” with visual cues




External tester: to be plugged in the device





Automatic CHECK procedure:
results either in ERROR or OK
response

Customisation for individual preference

- Choice between 2 sounds and 2 background display colours

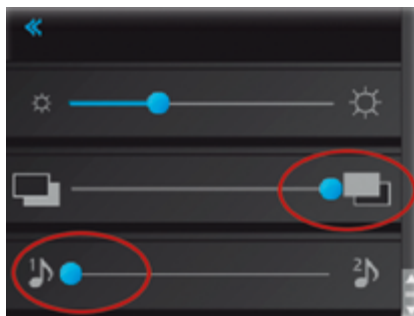

- Easy setting of optional Dr's Choice apical line

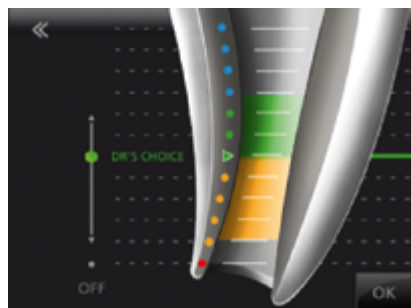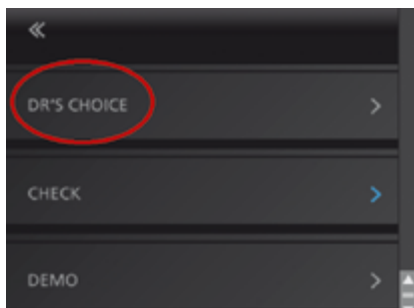

Range of products

REF

RAYPEX®6 Kit

1113 000 000

- 1 RAYPEX®6 apex locator
- 1 measuring cable
- 2 lip clips
- 2 file clips
- 1 touch probe
- 1 charger
- 1 tester for functional check

Accessories

- | | |
|------------------------|--------------|
| 5 lip clips | 0141 000 514 |
| 2 file clips | 0141 000 515 |
| 2 touch probes | 0141 000 518 |
| 1 measuring cable | 1119 000 506 |
| 1 charger | 1119 000 501 |
| 1 rechargeable battery | 1119 000 507 |