

Successful warm filling
in a few simple steps

Kindly note: Root canal must be prepared to a conicity of 5 % or higher and cleaned thoroughly.



**Adjust obturation
reference length**

Adjust obturation
reference length
with the help of a
stopper.

(= working length
- approx. 1 mm)

Select obturator

Insert GuttaMaster®
verifier into the root
canal. When tug-
back effect is achie-
ved at obturation
reference length,
select obturator in
the corresponding
colour.

Heat up obturator

Place obturator in
the holder of the
GuttaMaster® oven
and push the button
of the corresponding
obturator size.
Please ensure: The
silicon stopper must
be **underneath** the
holder!

Apply sealer

After having dried
the canal, apply
sealer thinly and
evenly with a paper
point (e.g. 2Seal) to
the root canal. Re-
move excess sealer
with another paper
point.

Fill root canal

Insert the Gutta-
Master® obturator
into the root canal
with steady, apically
directed pressure
and linger for a few
seconds with light
pressure.

**Separate core and
handle**

Remove the obtu-
rator handle with
a GuttaMaster®
PREPI® Bur (turbine
150,000-200,000
rpm without water
cooling).

**Remove excess
gutta-percha**

If necessary com-
pact gutta-percha
around the carrier,
e.g. with Machtou-
Plugger. Remove
excess gutta-percha,
e.g. with a warm
endodontic exca-
vator.

VDW GmbH

P.O. Box 830954 • 81709 Munich • Germany
Tel. +49 89 62734-0 • Fax +49 89 62734-304
www.vdw-dental.com • info@vdw-dental.com

